
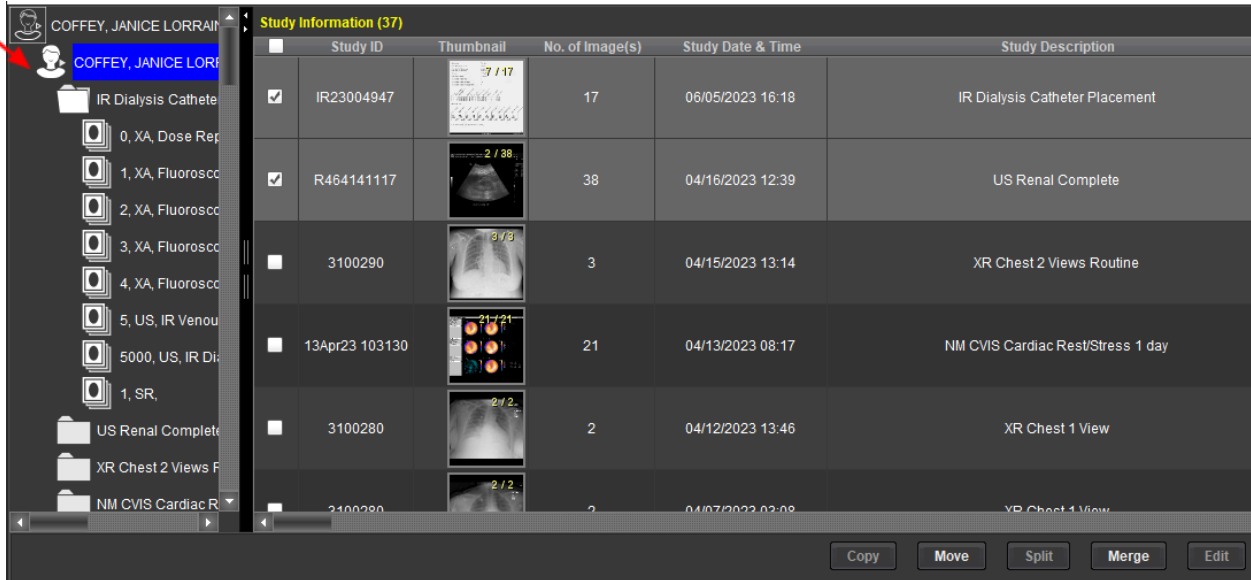





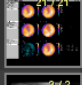

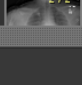
To Merge Exams:

1. Launch the QC Editor 
2. Select patient name on the left to view patient level (this will show you all of their exams)
3. Select the 2 exams you want to merge.
4. Click merge button below



COFFEY, JANICE LORRAINE


Study Information (37)

Study ID	Thumbnail	No. of Image(s)	Study Date & Time	Study Description
<input checked="" type="checkbox"/> IR23004947		17	06/05/2023 16:18	IR Dialysis Catheter Placement
<input checked="" type="checkbox"/> R464141117		38	04/16/2023 12:39	US Renal Complete
<input type="checkbox"/> 3100290		3	04/15/2023 13:14	XR Chest 2 Views Routine
<input type="checkbox"/> 13Apr23 103130		21	04/13/2023 08:17	NM CVIS Cardiac Rest/Stress 1 day
<input type="checkbox"/> 3100280		2	04/12/2023 13:46	XR Chest 1 View
<input type="checkbox"/> 3100280		2	04/07/2023 02:08	XR Chest 1 View

Copy Move Split Merge Edit

5. Use the *MERGE* arrow in the middle to choose which exam you will merge the studies into and click merge.

6.

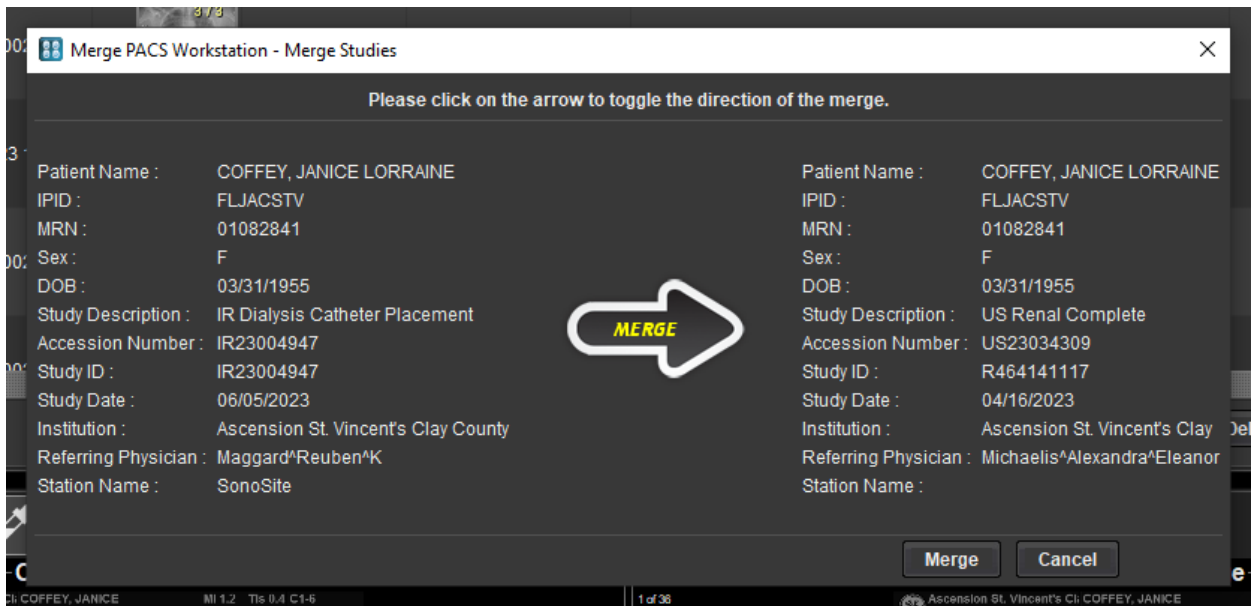

Merge PACS Workstation - Merge Studies

Please click on the arrow to toggle the direction of the merge.

Patient Name : COFFEY, JANICE LORRAINE	Patient Name : COFFEY, JANICE LORRAINE
IPID : FLJACSTV	IPID : FLJACSTV
MRN : 01082841	MRN : 01082841
Sex : F	Sex : F
DOB : 03/31/1955	DOB : 03/31/1955
Study Description : IR Dialysis Catheter Placement	Study Description : US Renal Complete
Accession Number : IR23004947	Accession Number : US23034309
Study ID : IR23004947	Study ID : R464141117
Study Date : 06/05/2023	Study Date : 04/16/2023
Institution : Ascension St. Vincent's Clay County	Institution : Ascension St. Vincent's Clay
Referring Physician : Maggard*Reuben^K	Referring Physician : Michaelis*Alexandra*Eleanor
Station Name : SonoSite	Station Name :

MERGE

Merge Cancel